
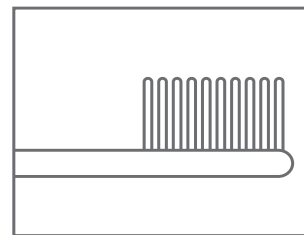
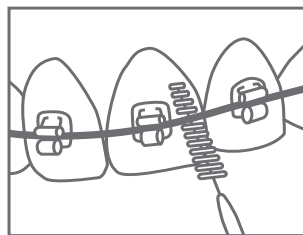
VivaDent® Fluoridation Gel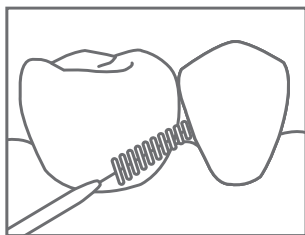

[en] Application Notes

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2024-11-05 / Rev. 1

1 Description

VivaDent® Fluoridation Gel is a remineralizing gel without any abrasive particles for gentle care, strengthening and protection of teeth and gums. It contains xylitol to inhibit bacterial growth, as well as D-panthenol (provitamin B5) to care for the gums and mucous membrane.

2 Ingredients

Aqua, Xylitol, Hydroxyethylcellulose, Alcohol, Potassium Fluoride, Calcium glycerophosphate, Laureth-23, Panthenol, Methylparaben, Aroma, Sodium saccharin

Fluoride content: 1450 ppm

3 Purpose of use

- For moisturizing gums and oral mucosa
- For cleaning interdental spaces in cases of increased caries risk
- For cleaning and maintaining the health of teeth
- For sensitive tooth necks
- In cases of sensitivity after tooth whitening
- For protection against caries and erosions

4 Recommendations for use

Tip: Spit out after use, do not rinse! This supports the caring effect of the product.

Use the product no more than twice a day according to your needs:

- Apply to the teeth after brushing.
- Apply to an interdental brush and clean interdental spaces, orthodontic appliances or fixed dental prostheses (e.g. bridges or implants).
- Use as an alternative to toothpaste for tooth brushing. Brush your teeth with your usual toothpaste at least once a day.
- Fill into a tray and wear the tray for 10 min (maximum once daily).

5 Warnings

- Before use, visually inspect the packaging and the product for damage. In case of any doubts, please contact Ivoclar Vivadent AG or your local dealer.
- Do not use if you are known to be allergic to any of the ingredients in this product.
- Contains potassium fluoride.
- Children of 6 years and younger: Use a pea sized amount for supervised brushing to minimize swallowing. In case of intake of fluoride from other sources consult a dentist or doctor.

6 Storage and handling instructions

- Storage temperature < 30 °C (< 86 °F)
- Observe the expiry date.

7 Additional information

Keep material out of the reach of children!

The manufacturer accepts no liability for damages resulting from improper use of the product. Furthermore, the user is responsible for determining the product's suitability for the user's own purposes before use.

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