VOCO

GrandioSO® Unlimited

EN

Instructions for use

MD EU Medical device

In accordance with DIN EN ISO 4049

CE

0482

Product description:
GrandioSO Unlimited is a light-curing, radiopaque nano-hybrid universal composite, which can be applied in layer thicknesses of up to 4 mm. **GrandioSO Unlimited** contains 91% by weight inorganic fillers and is applied with a dentine enamel bond. It is available in screw syringes and caps for direct application. The simplified shade system of **GrandioSO Unlimited** with its five Cluster-Shades covers all 16 VITA® classical shades.

Shades:
A1, A2, A3, A3.5, A4, Bleach and Opaque

- Indications:**
- Class I to V restorations
 - Reconstruction of traumatically damaged anteriors
 - Facetting of discoloured anteriors
 - Correction of shape and shade for improved aesthetic appearance
 - Locking, splinting of loose teeth
 - Repairing veneers
 - Restoration of deciduous teeth
 - Core build-up under crowns
 - Composite inlays

Contraindications:
GrandioSO Unlimited contains methacrylate and BHT. **GrandioSO Unlimited** should therefore not be used in patients with a known hypersensitivity (allergy) to these ingredients.

Patient target group:
GrandioSO Unlimited is suitable for use in all patients without any age or gender restrictions.

Performance features:
The product's performance features satisfy the requirements of the intended use and the relevant product standards.

User:
GrandioSO Unlimited should only be used by a professionally trained dental practitioner.

Use:

Preparation:
Clean the teeth being treated. If applicable, mark occlusal contact points. Bring material to room temperature before use.

Shade selection:
With the **GrandioSO Unlimited** Cluster-Shades A1, A2, A3, A3.5 and A4, the 16 VITA® classical shades can be covered as follows:

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Determine the shade on the cleaned tooth, before anaesthesia and application of a rubber dam, using a VITA® classical shade guide. Do this with natural daylight or room lighting as the light source. Operating theatre lights are unsuitable for shade selection, as their colour temperature is different from daylight. Based on the selected VITA® classical shade, the matching **GrandioSO Unlimited** shade can be easily chosen from the table.

Please note : Due to the special composition **GrandioSO Unlimited** shows its final colour and translucency only after light polymerisation. **GrandioSO Unlimited** can therefore not be used for colour selection without a completed light polymerisation.

Cavity preparation:
In principle, the cavities should be prepared according to the rules for adhesive restorative treatment; preparation should be minimally invasive in order to protect healthy dental hard tissue. For anterior restorations, angle off the enamel margins. Then clean and dry the cavity. Caries-free lesions in the cervical area do not need to be prepared; thorough cleaning is sufficient in this case.

Ensure that the working field is sufficiently dry. The use of a rubber dam is recommended. It is recommended that shaping aids be used in cases where a portion of the cavity is in the proximal region. It helps to use translucent matrices, wedged in the proximal region. Keeping separation to the minimum makes it easier to shape the proximal contacts and attach a matrix. Suitable protection for the pulp should be applied close to the pulp and covered with a layer of stable cement if necessary.

Bonding materials:
GrandioSO Unlimited is applied with a dentine enamel bond, using the adhesive technique. Any light-curing bonding materials can be used. Observe the relevant instructions for use during preparation (etching technique) and processing.

Application:
Apply the selected shade of **GrandioSO Unlimited** (see shade selection) in layers of no more than 4 mm thickness; adapt using a suitable instrument and then cure.

Light-curing:
Conventional polymerisation devices are suitable for light-curing the material. Depending on light output, polymerisation times for a layer thickness of 4 mm are as follows:

LED-/halogen lamp	A1, A2, A3, A3.5, Bleach	A4, Opaque
≥ 1200 mW/cm²	10 s	20 s
≥ 800 mW/cm²	20 s	40 s

Bring the light aperture as close as possible to the light-curing unit as otherwise the curing will be less effective. Inadequate curing can cause discolouration and complaints.

Finishing:
Restorations can be finished and polished immediately after removing the matrices (e.g. fine and extra-fine diamond burs, abrasive discs). The tooth should then be fluoridized.

GrandioSO Unlimited can be used for direct or indirect inlays using the known procedures. The physical stabilisation values are improved by the usual external (additional) curing.

- Warnings, precautionary measures:**
- Phenolic substances, especially products containing eugenol and thymol, lead to curing failures in restorative composites. Consequently, the use of zinc oxide/eugenol cements or other materials containing eugenol should be avoided in combination with restorative composites.
 - Each **GrandioSO Unlimited** Caps unit should be used for one patient only and should be applied exerting uniform but not excessive pressure.
 - Our information and/or advice do not relieve you of the obligation of checking that the products supplied by us are suitable for their intended purpose.

Constituents (in descending order according to content):
Ytterbium trifluoride, barium aluminium borosilicate glass, TCDDMA, BisEMA, BisGMA, initiators, stabilisers, pigments

Storage instructions and application method:
Storage at 4°C-28°C. Seal the syringes immediately after removing the material in order to avoid exposure to light and subsequent polymerisation. Do not use after the expiry date.

Disposal:
Dispose of the product in accordance with local regulations.

Reporting obligation:
Serious events such as death, temporary or permanent serious deterioration of a patient's, user's or other person's health condition and a serious risk to public health that arise or could have arisen in association with the use of **GrandioSO Unlimited** must be reported to VOCO GmbH and the responsible authority.

Note:
The Summary of Safety and Clinical Performance of **GrandioSO Unlimited** can be found in the European database on medical devices (EUDAMED – <https://ec.europa.eu/tools/eudamed>). Detailed information can also be found at www.voco.dental.

VITA classical A1-D4® is a registered trademark of VITA Zahnfabrik, H. Rauter GmbH & Co. KG, Bad Säckingen, Germany.

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