VOCO

VOCO Perfect Protect®

EN Instructions for use

MD EU Medical device

Product description:

The desensitising gel **VOCO Perfect Protect** is an adhesive, viscous gel with 3 % potassium nitrate and 0.11 % fluoride for the treatment of tooth hypersensitivity.

VOCO Perfect Protect is meant to be used in a whitening tray.

As part of tooth whitening, **VOCO Perfect Protect** can be used before and/or after the whitening treatment.

Indications:

Sealing of exposed dentine.

Contraindications:

- Not intended for use during pregnancy or breastfeeding.
- Not intended for patients who could swallow or aspirate the tray.
- **VOCO Perfect Protect** should not be used in patients with a known hypersensitivity (allergy) to one of the ingredients.

Patient target group:

VOCO Perfect Protect is suitable for use in all patients with permanent dentition without any age or gender restrictions.

Performance features:

The product's performance features satisfy the requirements of the intended use and the relevant product standards.

User:

The product is used by a professionally trained dental practitioner or is provided for use at home to the patient.

Use:

Preparation:

Clean the teeth being treated. Bring material to room temperature before use.

Application and cleaning:

1. The gel is applied with the help of a tooth whitening tray. If a tooth whitening tray is already existing, **VOCO Perfect Protect** can be used with this tray. Alternatively, a tooth whitening tray individually prepared by the dentist can be used.
2. Remove the cap from the syringe and keep it. Fill the gel as a strand into the cavities of the tray. A complete dental arch should use up to approximately 1/4 of a syringe. After extraction of the gel, close the syringe tightly with the original cap.
For better retention when treating a purely local sensitivity, a small amount of gel should be applied to the molar area of the tray to hold it in position.
3. Place the tray on the teeth and remove excess gel with a toothbrush or finger.
4. The duration and frequency of wearing the tray depends on the intensity of the tooth hypersensitivity and the dentist's recommendation. The treatment time can range from 15 minutes to one hour.
5. Immediately after removal of the tray rinse the mouth with water and remove the remaining gel with a toothbrush. Do not swallow when rinsing.
6. Clean the tray thoroughly with water and a toothbrush after wearing, dry it and then store it in a clean and dry place. No gel residue should remain in the tray.


Warnings, precautionary measures:

- If problems occur, discontinue treatment and contact a dentist.
- Excess material when placing the tray and residues after removal should be removed with a finger or brushed off and rinsed.
- DO NOT swallow material.
- No drinking or eating while wearing the tooth whitening tray.

Storage instructions and application method:

Storage at 4°C - 23°C.

Use the gel for a maximum of 14 days after opening the syringe. Do not use after the expiry date. Close the syringes with the original cap immediately after material extraction.

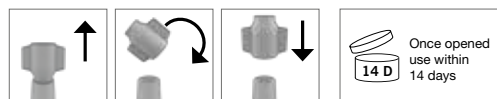Disposal:

Dispose of the product in accordance with local regulations.

Reporting obligation:

Serious events such as death, temporary or permanent serious deterioration of a patient's, user's or other person's health condition and a serious risk to public health that arise or could have arisen in association with the use of **VOCO Perfect Protect** must be reported to VOCO GmbH and the responsible authority.

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VC 60 BB1019 E1 0724 99 © by VOCO